


This page will be inserted into "FINESIA TL Implant (Prosthesis) System Manual" :
between page 21 and 22.

Reference

When the margin of the TL implant is set subgingivally, the pick-up impression is obtained using a direct impression cap, and the analog for a direct abutment is applied to the surface of impression. Thus, the model can be created. This allows precise reproduction of the position of the margin.

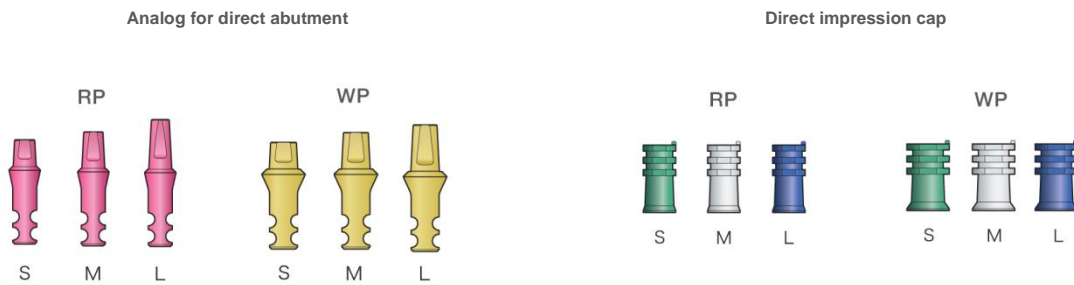

Notes

To take an accurate impression, ensure that the margins of the implant and impression cap are not damaged before use. When attaching the direct impression cap, align the orientation between the projection of the cap and the dimple in the abutment.

